

## Product Specification

[Product Name] Surgical Gown

This product has passed the factory inspection.

[Style, Size & Packing Specification]

Style	Size	Packing
Jumpsuit	M, L	1 set/ plastic bag

### [Product Description]

It is mainly made of 3 ply of non-woven fabric with breathable PE film laminate. Made by cutting and sewing.

### [Application]

1. Hospital operation and treatment of patients;
2. Epidemic prevention inspection in public places;
3. Disinfection of virus contaminated areas;
4. Also widely used in fields of military, medical, chemical, environmental protection, transportation, epidemic prevention, etc.

### [Features]

Anti-droplets, Waterproof, Anti-penetration, Breathable & Permeable

### [Notes, Warnings & Tips]

1. It is just for one-time use.
2. It is not allowed to be used if inner package is damaged.
3. It should be managed in accordance with the Medical Waste Management Regulations after using.

### [Storage and Transportation]

Store in room temperature with relative humidity no more than 80%, and at clean environment without corrosive gas. Transport by conventional means.

[Production Date] See label.

[Expiring Date] 2 years after production date.

### [Wearing Guide]

#### How to wear:

- \* Face the inside of the gown and put on the sleeves.
- \* Lean forward a little, tie the neckband.
- \* Stand up straight, and fasten the waist belt to fit your body well.



#### How to take off:

- \* Untie the back belt and neckband, grasp the right shoulder of the gown with the left hand, turn the sleeve outward, and same way to remove the left shoulder of the gown.
- \* Roll from top to bottom, put the contamination face inward until all is removed and place into the medical waste bag.



[Product Record Number] Sutong Medical Device Registration No. 20200044

[Last-Modified Date of this Instruction] Apr. 13th. 2020

[Medical Symbol Clarification]

